



# Home Instructions for Cleaning Invisalign Crystals

## **Warnings**

Do not place crystals in mouth or ingest. In case of accidental ingestion, immediately contact poison-control center. Keep this product out of the reach of children and pets. Do NOT use to clean on natural teeth, orthodontic appliances containing metal parts, jewelry or other objects. Avoid splashing cleaning solution on clothing and fabrics.

## **Aligner/Retainer Cleaning Instructions**

- Remove aligners / retainers and rinse them thoroughly under tap water.
- Place aligners/retainers into a cleaning container and fill with 100 ml warm water or to fill mark on tub.
- Add one packet of cleaning crystals.
- Gently shake/agitate for 20 seconds to dissolve and distribute crystals.
- Let stand for 15 minutes.
- Gently shake/agitate again for another 20 seconds.
- Rinse aligners/retainers and container thoroughly with warm water.

## **Storage**

- Store crystals in a cool, dry place.

**Andrew L. Reingold, D.M.D.**

General and Cosmetic Dentistry

59 East 54<sup>th</sup> Street, Suite 21, New York, NY 10022 - Tel: 212.724.4669 – [www.everlastingSmile.com](http://www.everlastingSmile.com)